

# LOCOMOTOR ULTRA SLIDE SHEET (tubular, washable) – USER INFORMATION SHEET



PRODUCT CODE	DESCRIPTION	DIMENSIONS
LOCO-001X	Standard	122cm* x 71cm
LOCO-002X	Long	145cm* x 71cm
LOCO-003X	Wide	122cm* x 100cm
LOCO-012	Compact	72cm* x 70cm
LOCO-013	Maxi	195cm* x 70cm
LOCO-067	Maxi Plus	195cm x 140cm*
LOCO-076	Mini	60cm* x 41cm
LOCO-115	Extra Long	200cm* x 71cm
LOCO-521	Extra Wide	200cm* x 140cm
LOCO-522	Large	200cm* x 90cm

\* Open edge

**IMPORTANT:** PLEASE RETAIN THIS USER INFORMATION SHEET WITH THE PRODUCT AND REFER TO IT BEFORE USE.

## DESCRIPTION & PURPOSE

The LOCOMOTOR ULTRA SLIDE SHEET is a fabric washable tubular slide sheet with a low friction inner surface allowing an individual to be repositioned without the need to lift. The LOCOMOTOR ULTRA SLIDE SHEET is available in various sizes and is used for a variety of repositioning tasks. The multidirectional glide of the fabric allows a patient to be moved in any direction and reduces shearing to the patient's body.

## INSTRUCTIONS FOR USE

Always ensure that any part of the patient not assisting with the transfer is on two layers of slide sheet.

### Patient turning

Using an approved rolling technique, position the LOCOMOTOR ULTRA SLIDE SHEET under the patient making sure that the hip and shoulder are over the slide sheet. Ensure central positioning to both bed and patient. To turn the patient take hold of the top layer of the LOCOMOTOR ULTRA SLIDE SHEET and slide the patient across the bed and then roll the patient back into the middle of the bed.

To remove the LOCOMOTOR ULTRA SLIDE SHEET, take hold of the bottom layer of the sheet and gently slide out from beneath the patient ensuring that one layer of slide sheet is sliding against the other. Steady the patient as this is performed.

**Moving a patient up the bed:** Position a LOCOMOTOR ULTRA SLIDE SHEET under the patient as described above, then, using an approved handling technique, take hold of the top layer of the LOCOMOTOR ULTRA SLIDE SHEET and slide the patient to the required position. Remove as above.

For information about sliding techniques contact your Moving & Handling Trainer, Back Care Advisor or equivalent.

**All patient moving and handling tasks should be carried out in accordance with approved patient moving and handling techniques and after a full assessment of the patient, equipment, task and environment.**

**Always follow the approved policy of your place of work when adopting moving and handling practices.**

**INSPECTION:** Before each use inspect the product for wear and damage. **Do not use** if the product is showing any signs of wear or damage.

**STORAGE:** Store in dry conditions away from direct sunlight and sources of heat.

**MATERIAL:** Silicone coated nylon.

**LAUNDERING:** May be washed at 71°C and can also be autoclaved. May be sterilised using Sodium Dichloro-cyanurate or Sodium Hypochlorite – wipe only and rinse with water.

**DISPOSAL:** Please refer to the policy of your place of work for the disposal of non-hazardous waste.

For more information please contact your supplier or contact customer services at:



Select Healthcare (UK) Ltd., Unit 2 Albert Road, Rushden, Northants, NN10 0BU  
Tel: (01933) 411729 Email: info@selecthealthcare.co.uk  
Website: www.selecthealthcare.co.uk

